



GIRAFFE
BOTTLE

**ASSISTIVE TECH
DRINKING SYSTEM**

CARE & USE GUIDE

...FEATURING THE JOURNEY NECK



SCAN ME



**TRANSLATIONS
AVAILABLE**

ES FR IT DE

WWW.GIRAFFEBOTTLE.COM/LEARNING-CENTER

Components

Tritan™ Plastic Bottle

INCLUDES:

- SPOUT LID + CAP



SAFE FOR
DISHWASHER



Stainless Steel Bottle

INCLUDES:

- SPOUT LID + CAP
- BOTTLE CLEANING BRUSH
- INSULATED HANDLE LID




NOT SAFE FOR
DISHWASHER



Spout Lid, Cap + Silicone Parts


- ! Silicone parts should be removed from lid before dishwasher cleaning

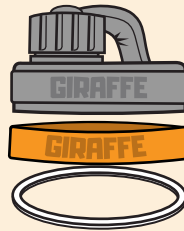

SAFE FOR
DISHWASHER

Journey Neck Components

- DRINKING TUBE
- BITE VALVE
- TUBE CLEANING BRUSH

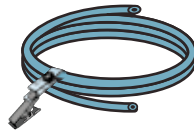

HAND-WASH
ONLY




HAND-WASH
ONLY



For optimal hygiene/function,
all bottle components should be
cleaned before each use.

Carry Bag + Bottle Holder

Take your Giraffe Bottle on the go. Look into our optional carrying accessories for a more hands-free experience.

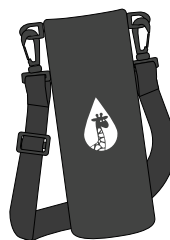

Journey Neck Use + Assembly

Bite down on the valve to allow flow of liquid, and release to close. You can use the clip to attach to clothing as needed. Assembly should include:


- DRINKING TUBE WITH LID ADAPTER
- THREADED SPOUT LID WITH GASKET
- METAL CLIP
- BITE VALVE



DO NOT PLACE BOTTLE IN FREEZER



TUBE CAN BE CUT TO SIZE IF NEEDED

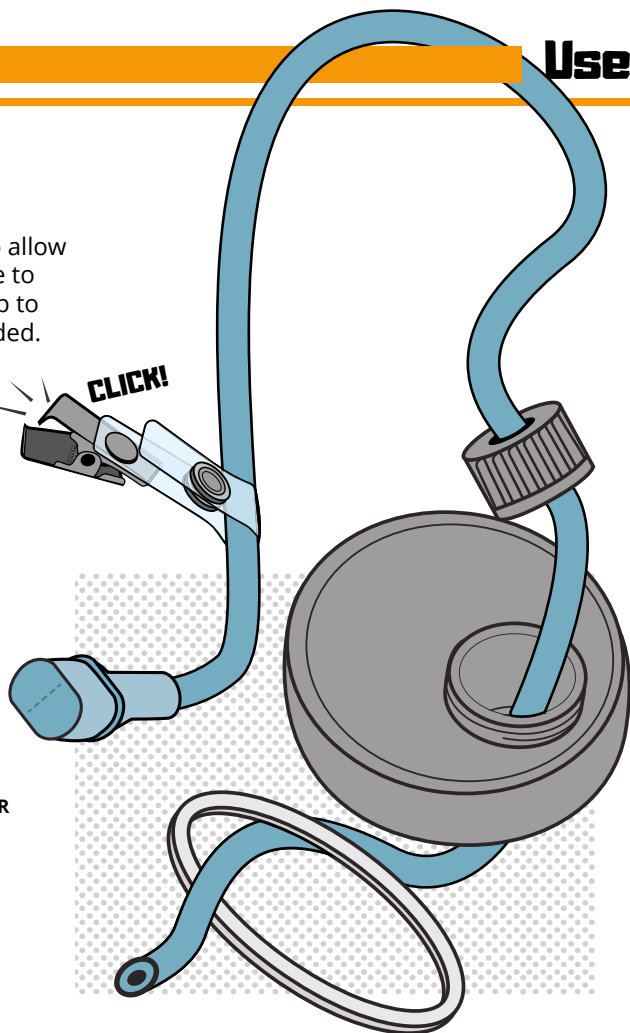

Caution: Bite Valve is removable; this system is recommended for individuals with good control over mouth functions

Care


Drinking Tube + Bite Valve

The drinking tube and bite valve should be hand-washed, and replaced regularly.

USE INCLUDED
CLEANING BRUSH



NOT SAFE FOR
DISHWASHER

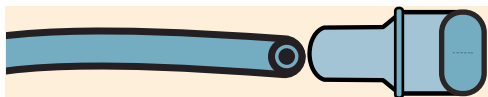

Drinking Tube Maximum
Operating Temperature: 160°F / 71°C

Giraffe Bottle Accessories



Replacement Giraffe Bottles

Our Giraffe Bottles come in a few sizes, materials, and colors. Dress up your drink with a bottle that fits you best!

Tritan Clear: #504111

Smoke Black: #504211

Burnt Orange: #504212



Rail Mount Bottle Holder

A bottle holder perfect for powerchairs with accessory rails.

#500314



Bottle Carry Bag w/ Strap

A bag great for carrying bottles on shoulders, or looped over a chair.

#504062



Clamp Mount Bottle Holder

Our adjustable holder mounts to wheel-chairs, hospital beds, or other common frames.

#500311



Drinking Tube w/ Bite Valve + Clip

Our drinking tube and bite valve combo. Easily cared for with a separate tube cleaning brush.

Tube + Valve: .. #505181

Cleaning Brush: #504712



Different Levels of Hydration

Choose from different assistive hydration methods featuring our Journey Drinking Tubes and Tower Necks (original and XL).

More Accessories Online

WWW.GIRAFFEBOTTLE.COM

SALES@GIRAFFEBOTTLE.COM

1-630-922-5010



a division of:
Oak Hill
BRANDS CORP.

Product designed in the USA.
Manufactured with domestic and foreign components.
Information and specifications subject to change at any time without notice.
Giraffe Bottle is a registered trademark of Oak Hill Brands Corp.