



GIRAFFE[®]
BOTTLE

HYDRATION WITH A PURPOSE[™]

**HANDS FREE
DRINKING SYSTEM**

CARE & USE GUIDE



FOR THE
**TOWER XL
NECK**

**REGISTER
YOUR
PRODUCT**



FOLLOW US!

WWW.GIRAFFEBOTTLE.COM/REGISTER

Components + Care

Tritan™ + Stainless Steel Bottle Kits




DO NOT PLACE BOTTLES IN FREEZER/MICROWAVE

Recommended Max Temperature: 185°F / 85°C



SAFE FOR DISHWASHER

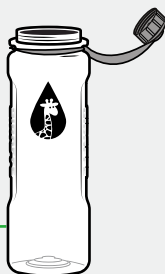

TRITAN BOTTLE



TRITAN LID



STAINLESS LID




NOT SAFE FOR DISHWASHER



HAND-WASH ONLY



STAINLESS BOTTLE

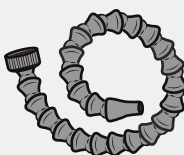

MODULAR NECK



DRINKING TUBE




BITE VALVE




Avoid submerging modular neck during cleaning—wipe down outer surface.

Drinking Tube + Bite Valve

The bite valve must be removed from the drinking tube during cleaning and should **NOT** be pierced with a brush.

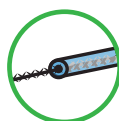



USE INCLUDED CLEANING BRUSH

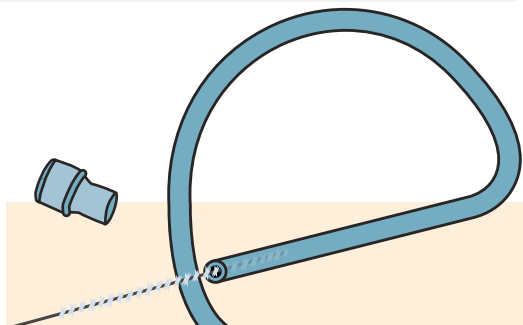

If you have a taste sensitivity to the supplied drinking tube (after washing), inquire for an alternate tube.

TOWER XL Neck Assembly

The assembly is designed to **POP** apart to adjust neck length as needed.

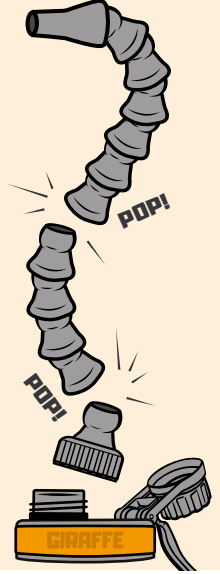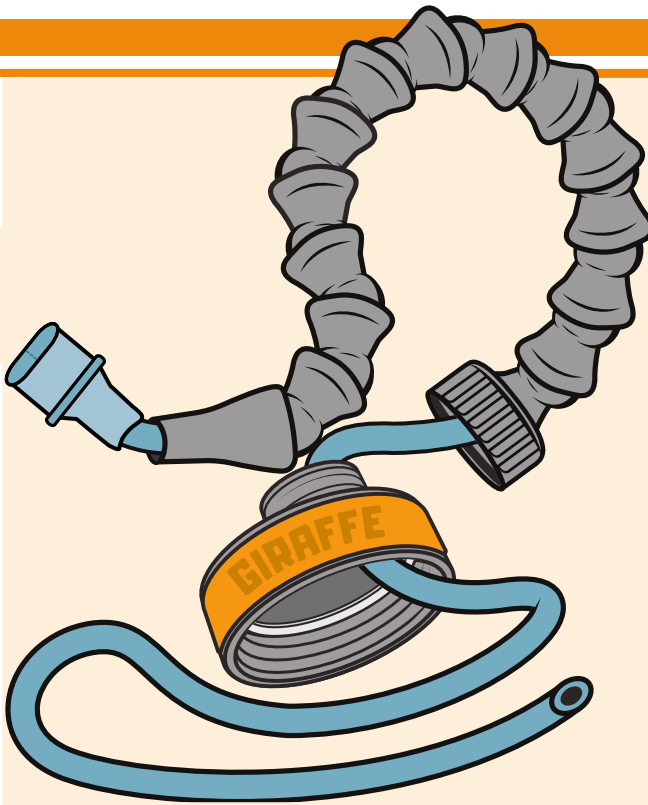

If you are concerned the bite valve will present a hazard as a small object, remove from the drinking tube during use

1.




2.




3.



1.

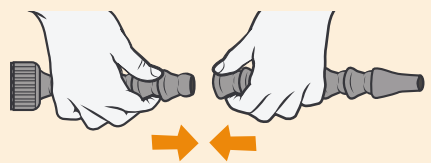

2.

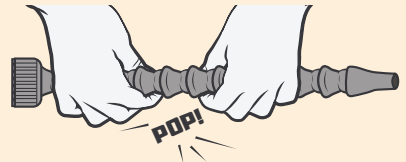

REPLACEMENT SCHEDULE



DRINKING TUBE MONTHLY

CLEANING BRUSH . . . EVERY 3 MONTHS

BITE VALVE MONTHLY

NECK ASSEMBLY . . . EVERY 8 MONTHS

Replace integral parts of the Giraffe Bottle Tower XL System regularly to maintain optimal bottle hygiene and function.



Recommended Use

Which Beverages Can You Drink?

All Giraffe Bottle systems are primarily recommended for **water and other non-viscous liquids**. Remove the bite valve when enjoying other beverages.



Ideal Use: The bite valve featured in the Tower XL system is primarily designed for **thin non-viscous drinks** such as **water, coffee, tea or soda**.




Use with Extra Care: You can use flavored waters, sports drinks, and juice, but excess sugar may reduce product lifespan. After use, rinse the straw and bite valve thoroughly. Remove bite valve from the tube before cleaning; don't pierce it.



Not Recommended: Using viscous or thicker types of liquids can clog the straw or bite valve. We don't recommend using Powdered Drinks, Ice-blended Beverages (ICEEs, Frappuccinos, etc), Juices with Pulp, Shakes or Smoothies.

Control the Flow

Pressing the **Bite Valve** opens it, allowing controlled drink flow with suction, and prevents leaks. It also stops liquid from retreating into the bottle. Easily removable if not needed.

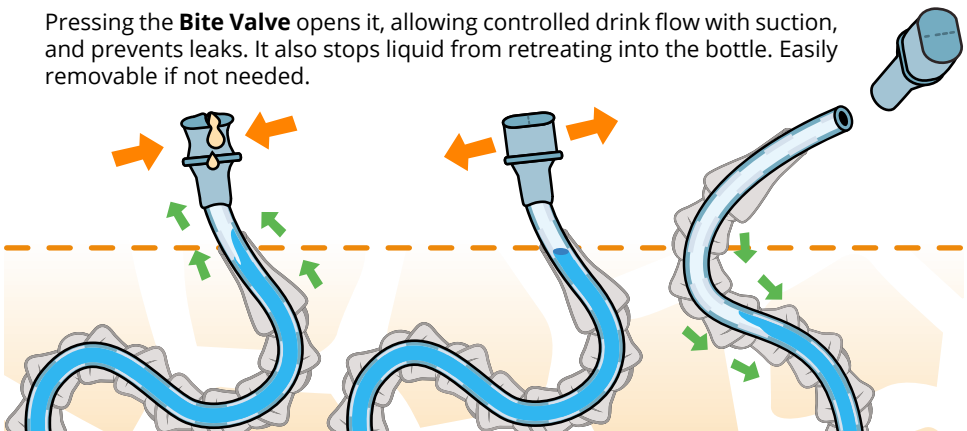

! This system is recommended for individuals with good control over mouth functions

THE TOWER XL SYSTEM






NECK LENGTHS AVAILABLE	15", 25", 30", 36" (38 cm, 64 cm, 76 cm, 91 cm)
DRINKING TUBE TYPE	Large, Housed in Modular Plastic 1/4" (6.4mm) Inner Diameter
FEATURED LIQUID CONTROL	Bite Valve *Removable from Tube
BOTTLE HOLDER OPTIONS	Clamp Mount, Rail Mount, Under Cushion, Carry Bag & Desktop
THIS SYSTEM IS BEST FOR:	Individuals with full lung and mouth function, but little-to-no upper body mobility.

LEVELS OF

HYDRATION

Giraffe Bottle offers a few unique options for Hands-Free Hydration. Each one is tailored for different needs and purposes, with experiences of individuals in mind.

Our systems are cross compatible with every bottle; explore them all and see which fits your needs best!

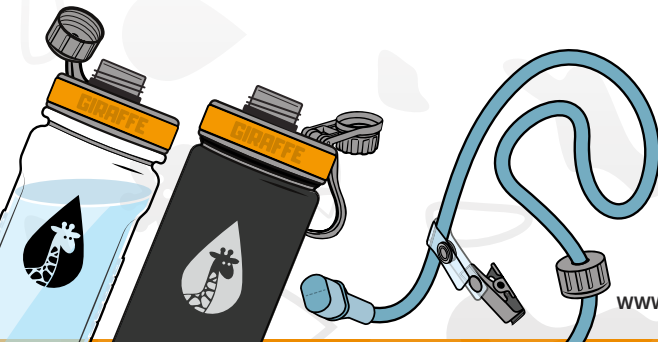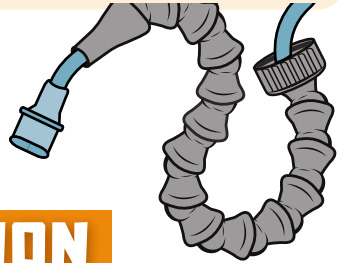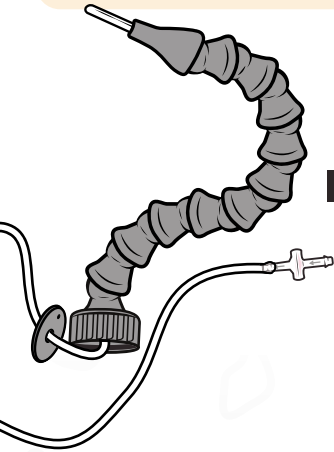

**VISIT
FAQ'S**

Questions about your
current system?

WWW.GIRAFFEBOTTLE.COM/PRODUCT-FAQ

LEARNING CENTER

Check out the Learning Center on GiraffeBottle.com to explore Hands-Free Hydration, and Frequently Asked Questions about our products. Feel free to contact Giraffe Bottle with any questions, comments, or feedback.



Giraffe Bottle Accessories



Replacement Giraffe Bottles

Tritan Clear: #504111
Matte Black: #504231
Burnt Orange: #504212
Ltd. Ed. SGN Pattern: #504221



Drinking Tube + Bite Valve

For 15" Neck: #504072
For 25" Neck: #504074
For 30" Neck: #504075
For 36" Neck: #504076



Tower XL Starter Kit

15" Neck: #505162
25" Neck: #505164
30" Neck: #505165
36" Neck: #505166



Clamp Mount Bottle Holder

Mounts to wheelchairs and other round tubing.
#500311



Cushion Companion Bottle Holder

Slide under mattresses, seat cushions, and other surfaces.
#500332



Weighted Desktop Base

Keep bottles secure and upright on flat surfaces with this sturdy bottle holder.
#500323



Rail Mount Bottle Holder

Attaches to powerchairs with accessory rails (includes Quantum™, Permobil™).
#500314



Bite Valves, Pack of 3

3 replacement silicone bite valves for the Tower XL and Journey systems.
#504672

More Available Online

For alternate drinking tubes (due to sensitivity in taste), accessories, and replacement parts, explore online or contact us.

WWW.GIRAFFEBOTTLE.COM



GIRAFFE
BOTTLE

a division of:
Oak Hill
BRANDS CORP.

SALES@GIRAFFEBOTTLE.COM

1-630-922-5004

Product designed and assembled in the USA. Manufactured with U.S. and global components.

Information and specifications listed are subject to change at any time without notice.

Permobil, Quantum, and all other brands that may have been mentioned, either directly or indirectly, are trademarks of someone else and are not the property of Oak Hill Brands. Giraffe Bottle is a registered trademark of Oak Hill Brands Corp.